



MEDICAL MAINTENANCE EQUIPMENT TIP SHEET

Medical Device Nomenclature

Monitor Patient Vital Signs, Portable

LIN: M66558 **NIIN:** 01-668-7223 **Model:** 203-2251312-01 (PROPAQ M)

Manufacturer: ZOLL Medical Corporation

User and Maintainer Resources (*Medical Materiel Information Portal*)

<https://a01.usamma.amedd.army.mil/mmip/Catalog/Item/016687223>



Representative Photo Only

Types of Organizations Who Have the Device:

- Hospital Center
- Forward Resuscitative and Surgical Team
- Medical Detachment, Veterinary Services
- Combat Support Hospital
- Special Forces Group

Level of Support: Field

- Operator cleaning, start up self-test, and accessory inspections
- Inspection, preventive maintenance, system output verifications, and validation of system calibration
- Medical device repair will be to the assembly or sub-assembly level.
- Modules and printed circuit boards (PCB) will be repaired by replacement.

Level of Support: Depot

- Depot is the only level authorized to perform module and PCB repair.
- All overhauls and rebuilds must be authorized by the U.S. Army Medical Materiel Development Activity's Medical Device Assembly Management, Program Management Office.

Startup List:

https://a01.usamma.amedd.army.mil/mmip/Content/filestore/Start-Ups/6515-01-668-7223_MONITOR%20PATIENT%20VITAL%20SIGNS%20PORTABLE_HANDBOOK_INSERT_STARTUP_070219.pdf

Field Maintenance Tips:

- To avoid battery pack damage, do not leave depleted for more than 14 days, and remove if being stored for more than 30 days. Battery will recharge to 100 percent in four hours from a low battery indication.
- Battery recalibration is needed when Recalibration LED icon (🔍) illuminates for approximately 10-seconds after the display button is pressed and released. Refer to the user manual for battery recalibration steps.
- 🔍 symbol is displayed in top right window if device is ready for use, patient monitoring is functional, and battery is charged.
- If an error has occurred, the ❌ symbol displays in top right window. See user manual for troubleshooting steps. If the error can't be resolved, contact the Biomedical Equipment Specialist.
- This medical device should not be stored or used outside of specified environmental temperature limits (22°F to 158°F).
- Do not sterilize monitor/accessories (unless labeled otherwise). Avoid abrasives including paper towels on monitor display.
- Use of this medical device in an aircraft can interfere with invasive blood pressure and carbon dioxide monitoring.

Warranty Information:

- Period: Five years for medical device; one year for battery
- Coverage: Defects in material and workmanship
- Notes: Does not apply if device has been repaired/modified without ZOLL authorization or if damage is attributed to incorrect (off-label) use, negligence, or accident.

POC: M2PA, Army Medical Logistics Command: usarmy.detrick.amlc.mbx.m2pa@mail.mil

